Research & Development Healthcare  
Europe, Middle East, Africa  
Microbiological Support

## **Expert's statement on the compatibility of Incidin premium wipes with the hard surface disinfectant Incidin foam**

### **Summary**

Incidin premium wipes were impregnated with Incidin foam (undiluted) in the designated Incidin wipe dispenser. After storage at ambient conditions over 4 weeks the concentration of active ingredients and the pH of the disinfectant solution expressed from these wipes was examined. A loss of <10% of active ingredient and no significant change in pH were observed.

We therefore recommend the following contact times for Incidin premium wipes impregnated with at least 2.5 l Incidin foam use solution in the dedicated dispenser to achieve bactericidal and yeasticidal as well as tuberculocidal hard surface disinfection in the healthcare area under clean and dirty conditions, while the impregnated wipes may be stored at ambient conditions for up to 4 weeks:

- Incidin foam undiluted with a contact time of 5 min under clean and dirty conditions

As in any surface disinfection process it is crucial that the entire surface is fully wetted with the disinfectant solution during the wiping procedure.

## Background

Ready to use wipes are a convenient way for disinfection of frequent touch surfaces in healthcare settings. However, it has been documented in the literature that prolonged contact of disinfectant use solutions with wipes may lead to adsorption of active ingredients to the wipe material<sup>1)</sup>. This may lead to decreased disinfectant efficacy, when using such wipes. Aim of this study was therefore to provide data on the concentration of active ingredients in Incidin foam solution after a 4 week contact with Incidin premium wipes.

## Materials and methods

Incidin premium wipes were impregnated with 2.5 l of undiluted Incidin foam. The impregnated wipes were stored at ambient conditions for 4 weeks. The fresh solution and the stored samples were used to express the disinfectant from the wipes. The expressed liquid was then submitted for analytical investigation.

The pH of the solution was detected with a glass electrode. The active ingredients ethanol and 2-propanol were determined by gas chromatography. Benzalkoniumchloride was determined by potentiometric titration and Glucoprotamin by liquid chromatography.

Test were run at Henkel AG & Co. KgaA Corporate Analytics and are documented in this laboratory under reference 11-06910.

## Results

In the following table analytical results of the fresh solution and of the solution expressed from the wipes after 4 weeks of storage are given (any percentage is given as weight/weight):

| Parameter            | Fresh solution | Expressed from wipes |         | % change |         |
|----------------------|----------------|----------------------|---------|----------|---------|
|                      |                | 1 week               | 4 weeks | 1 week   | 4 weeks |
| pH                   | 7.87           | 8.10                 | 8.10    | n.a.     | n.a.    |
| 2-propanol           | 19.3%          | 18.6%                | 18.2%   | -3.6     | -5.7    |
| Ethanol              | 9.5%           | 9.2%                 | 9.0%    | -3.2     | -5.3    |
| Benzalkoniumchloride | 0.19%          | 0.20%                | 0.20%   | +5.3     | +5.3    |
| Glucoprotamin        | 0.0072%        | 0.0069%              | 0.0070% | -4.0     | -2.5    |

n.a. = not applicable

## Discussion

A change in analytical parameters of <10% is used as a pass criterion in this evaluation. This can be regarded to cover the range of precision of such analytical tests. Also a loss of <10% of active ingredient is regarded to have no significant impact on the efficacy of disinfectants.

Applying these criteria the Incidin foam solution expressed from Incidin premium wipes is evaluated as effective with no significant change of analytical parameters.

## Conclusion

The use of Incidin premium wipes impregnated with at least 2.5 l of Incidin foam (undiluted) in the designated Incidin wipe dispenser is regarded appropriate to achieve surface disinfection with bactericidal and yeasticidal as well as tuberculocidal efficacy spectrum at a contact time of 5 min, when the wipes are stored at ambient conditions for up to 4 weeks.

The amount of use dilution used on a surface has been demonstrated to have an influence on the efficacy of disinfectants in practice<sup>2)</sup>. As in any surface disinfection process it is therefore of outmost importance that the entire surface is fully wetted with the disinfectant solution during the wiping procedure, when using Incidin premium wipes.

Düsseldorf, November 17<sup>th</sup> 2011

Dr. L. Geret  
*Director RD&E HC*  
*Research & Development Healthcare*

Dr. Bernhard Meyer  
*Manager Microbiological Support*  
*Research & Development Healthcare*

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*[podpis]*

Dr. L. Geret

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*Research and Development Healthcare*

*[podpis]*

Dr. Bernhard Meyer

*Manažer mikrobiologické podpory*

*Research and Development Healthcare*

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Praha, 16.9.2016

### Interpreter's clause

As a sworn interpreter of **English and Russian**, appointed by the decision of the Regional Court in Prague on **November 7, 2001**, under file No. Spr. **4108/2001**, I confirm hereto that the present translation corresponds in full to the original language of the attached document.

This translation is registered in the translator's records under No. 988/2016.

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