

# INSTRUCTION FOR USE WITHDRAWAL CANNULA



## This instruction for use is not part of products and is issued on customer request

- COROSPIKE GREEN ref: CPHF-CS01
- COROSPIKE BLUE ref: CPHF-CS05
- COROSPIKE RED ref: CPHF-CS10

Read the instructions carefully.

### Indications for Use:

GREEN type: easy withdrawal and supplementary injection, reliable protection against microbial contamination by integrated air filter 1,2 µm.

BLUE type: reliable protection against particles during withdrawal by 5 µm solution filter and with integrated air filter 1,2 µm

RED type: reliable protection against particles during withdrawal by 5 µm solution filter and with integrated air-vent filter 0,2 µm which is reliable handling of very sensitive solution for contamination and by protection against toxic aerosols

**Cautions:** Do not clean or re-sterilize. This device remains sterile until the package is opened or damaged. Do not use if the sterile package has been opened or damaged.














**Re-sterilization:** Product is not intended for re-sterilization. Manufacturer prohibits re-sterilization.

**Dangerous substances:** If unit package is marked with symbol "PHT-DEHP" product contains substances called dangerous. Corospiques do not contain dangerous substances.

**How to use:** Fully tighten all connection before use. Use aseptic techniques.

1. Open spike if packing not damages
2. Put spike in bottle
3. Open clips-vent by spike and connect syringe with spike
4. Draw liquids through Luer-Lock connection on spike to syringe
5. Disconnect syringe with spike and close clips-vent
6. Re-connect syringe with spike according to medical/hospital rules.
7. Destroy spike when disconnecting with bottle. Throw away.

### Used symbol on blister

 Product reference	 For single use only	 Sterilized by Ethylene oxide
 Batch/Lot number	 Do not use if unit package is damaged	
 Date of Manufacturing	 Keep away from sunlight	
 Use by/Expiry date	 Keep away from rain	
 Manufactured by	 Pyrogen free	
 See instruction for use	 Temperature limitation	



CliniPharma s.r.o., Viklefova 1795/10, 130 00 Prague , Czech republic, info@clinipharma.cz