CVC, HEMODIALYSIS AND ARTERIAL CATHETER SETS

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EN **Indications:** C.V.C.: Central Venous Catheter sets: Designed for short-term* central venous catheterization use in jugular, subclavian or femoral veins. Hemodialysis catheter sets: Designed for short-term* vascular access for Hemodialysis and Aphaeresis. Arterial Catheter Sets: Designed for short-term* use in radial, brachial and femoral arterial catheterization.

Contraindications: Distorted local anatomy (eg: vascular injury, prior surgery, radiation history), infection at insertion site.

Instructions for insertion of catheters using Seldinger technique: The attending Physician must be familiar with the Seldinger technique.

- 1.Prepare puncture site according to hospital procedure.
- 2.Administer local anesthetic, if necessary.
- 3.Flush guide-wire with heparin through the protecting tube, if desired.
- 4.Puncture the blood vessel using a Seldinger needle.
- Δ Do not apply force on the needle cannula as bending may occur.
- 5.Aspirate until good blood flow from a vein or pulsing from an artery is established.
- 6.Using straight Seldinger needle: disconnect the syringe and close the needle to prevent further bleeding or air embolism. Advance wire gently into vessel through the needle**.
- Using the Y needle: connect a syringe through the female port and advance the wire gently through the side valve port**.
- Using a Seldinger syringe: connect the syringe to a Seldinger needle and advance the wire gently through the hollow plunger**.
- CAUTION! Never withdraw the guide wire against the needle bevel. Damage or severing to the guide wire may occur. In wire manipulating through tortuous passages a kink in the wire can occur.
- 7.Hold the guide wire firmly in place, and carefully remove needle.
- 8.In cvc and dialysis catheter sets, using for enlarging the puncture site, vessel dilators can be used as well as a scalpel.
- Δ When using a scalpel, position cutting edge away from the guide-wire.
- 9.Remove dilator.
- CAUTION! Never leave the vessel dilator in place as an indwelling catheter.
- 10.Advance catheter over guide wire with a twisting motion up to the final position.
- 11.Hold the catheter firmly in place, and carefully remove the guide wire.
- 12.Confirm catheter placement by aspirating.
- 13.Secure the CVC catheter position with suture wing and a clip. For Hemodialysis and Arterial, use the suture wings on the hub.
- 14.Dress puncture site per hospital protocol.
- 15.For CVC and Hemodialysis sets: close off all unused extensions using the clamps.

WARNING!

- 1.Always exercise care during use or removal to prevent air aspiration into the blood vessel.
- 2.All catheters should be periodically inspected for integrity, flow rate, security of dressing and placement.
- 3.Withdraw catheter in any sign of occlusions, leaking, inflammatory and infection signs in the indwelling site.
- 4.Periodically check and secure all catheter connecting elements along the connecting line.
- 5.Never use a sharp instrument while removing the catheter dressing.
- 6.Do not suture directly to the outside diameter of the catheter to avoid cutting or damaging the catheter or impeding catheter flow.
- 7.While cleaning the indwelling site, do not expose the catheter tubing to solutions containing acetone or alcohol.
- 8.The device should be disposed after use in accordance with the hospital policy for biological waste.
- *Short term: less than 30 days According to Annex IX of the MDD.
- **If a "J" type of wire is used, straighten tip and advance the wire.

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